



BIG PAW



Making wheelchair life safer

TEMPLATE INSTRUCTIONS



To determine the correct sized BIGPAW for your rigid wheelchair, print off the template and follow the instructions.

BIG
PAW


1 Print the two page template (A & B) onto A4 paper

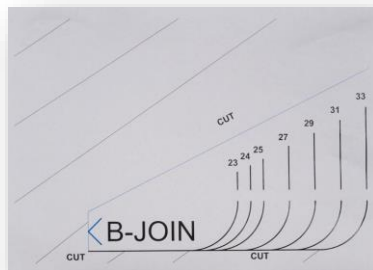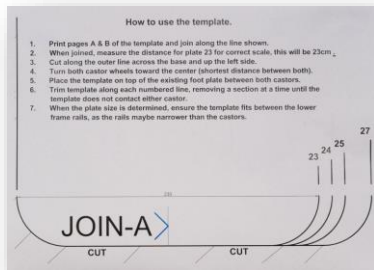

2 Measure the width of the smallest plate (23) on the template, this needs to be 23 cm for correct scale. If you require a template wider than 27, join page A to B and measure for correct scale.

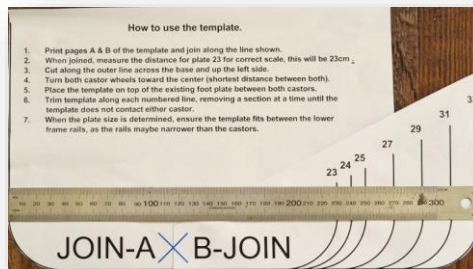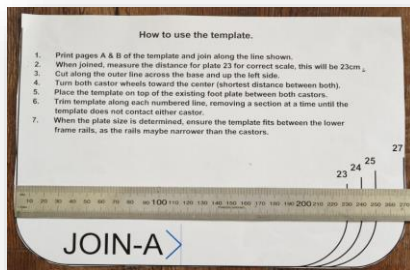

- 3** Cut along the 27 template line down both sides and along the bottom.

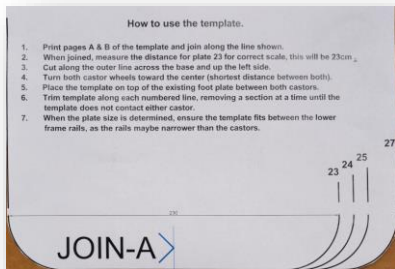


- 4** Turn the castors in towards each other, creating the narrowest distance





- 5** Place the template on the existing foot plate, in between the centre of both castors.

If the template contacts the castors, trim the template width down one size using a pair of scissors. In the image, the 27 template contacts each castor, trim the template to 25 and re fit between the castors.

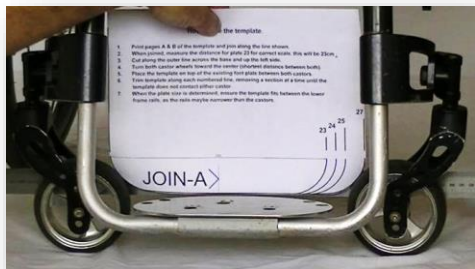

BIG
PAW

6 Refit the 25 template between the castors, now there is clearance between the template and each castor. Make allowance for some castor wear as there maybe a need to go to the next down sized template. Castors are usually 10cm (4") in diameter when new.

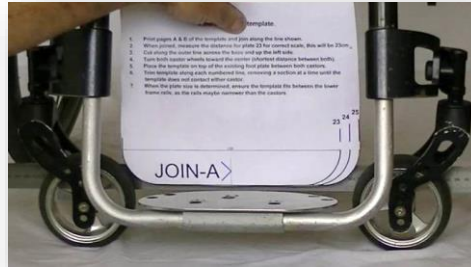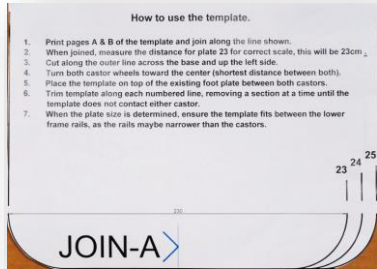

7 Once you have determined the correct size, you can rest assured knowing your BIGPAW will fit perfectly, providing increased protection, support & control for your feet.



Visit www.bigpaw.com.au for additional information and attachment tutorials



bluedog@git.com.au